INSTRUCTION FOR USE

THERMOSAVER

Hypothermic Rescue Bag

ENG

Please read all instructions before use.

These instructions for use describe uses of the Thermosaver Hypothermic Rescue Bag. Users must read these instructions for use carefully prior to using the Thermosaver for the first time so that all features are thoroughly understood. Please keep these instructions for use for future reference.

Indications for use

Patients should be over 5 years of age. Patient is suffering or likely to suffer from hypothermia.




Contraindications

Not to be used by children under 5 years. Not to be used on multiple patients.

Classification

The Thermosaver is classified as Class I according to the Medical Devices Regulation (MDR 2017/745).

Thermosaver only accepts responsibility for the equipment's safety, usability, and performance if - The equipment is used in accordance with its intended use, and - The equipment is used in accordance with the product documentation

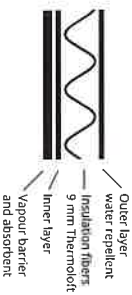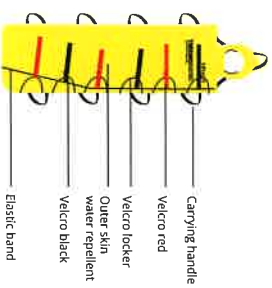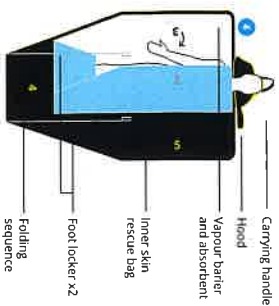Intended Use
The Thermosaver is intended to be used on patients suffering from or likely to suffer from hypothermia and stops heat loss. The device is intended for multiple patient multiple use.
Intended User
The intended user is the trained first responder familiar with the content of the Thermosaver's Instruction Manual. The device is not intended for lay-users.

Patients	Applies	Does not apply	Description
Neonatal		x	
Children	x		
Adults	x		The device is not intended for persons under 5 years
Disabled persons	x		
Operators		Applies	
Health Care Professionals*	x		
Certification required		x	The RU is sufficient to ensure proper use.

*Operators are trained first responders (e.g. emergency services, police or fire brigade officers) as well as volunteers with land or sea rescue units.

Operational Safety

The Thermosaver should only be operated by trained professionals who have read the instructions for use. Such personnel are referred to as operators throughout this manual. Do not expose to open fire.

Limitations on Reuse

Maximum number of allowable reuses is 9. The Thermosaver can be used up to 10 times subject to replacement of vapour barrier, cleaning, disinfection and drying following appropriate storage and transport of the bag according to 55-042 Processing Instruction. <https://thermosaver.no/produkt/instruktion-for-usage/>

Safety and adverse events

Operator must ensure that the instructions are followed appropriately to avoid an unlikely event of suffocation.

Safety regulations

Dispose of the used product and parts in accordance with accepted medical practice and applicable local and national regulations. Used product may be contaminated and requires a potential biohazard. The Thermosaver has a shelf life of 5 years and an intended lifetime of 10 years. The product should be replaced that has reached its end of life. The device should be reported to the manufacturer and the competent authority of the Member State on which the user and/or patient is established.

Environmental and Handling Conditions

Operation temperature range	-40°C to +50 °C
Storage temperature range	-40 to +70 °C
Maximum relative humidity for storage	95%

Symbols used on labels

Symbol	Meaning	Symbol	Meaning
	Manufacturer name and address		Unique device identifier (UDI Carrier)
	Serial Number		Product's a medical device
	Model number		Country of Origin
	Keep away from open fire		Link to RU
	WARNING: use return bag		

Package and Device Inspection

The Thermosaver should be inspected before use. - Ensure packaging is intact. Do not use if it appears to be damaged. - Ensure that the Thermosaver is within its expiration date. - Visually inspect the Thermosaver for any signs of damage to the bag itself. Do not use the Thermosaver if it appears to be damaged.

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

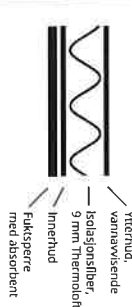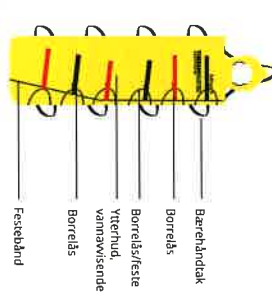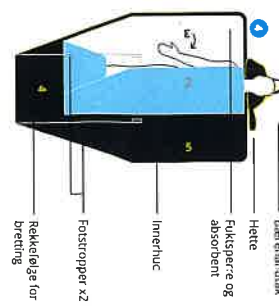
Contact information

Thermosaver AS
Ramsstadøkka 1
1850 Mysen
Norway

The training video is available on www.thermosaver.no and <https://thermosaver.no/produkt/instruktion-for-usage/>. If you have any questions or concerns related to this manual or Thermosaver devices, contact us at post@thermosaver.no

Please refer to the electronic RU at <https://thermosaver.no/produkt/instruktion-for-usage/>

Date of Issue: 01.09.2023

Kontaktinformasjon

Thermosaver AS
Ramsstadøkka 1
1850 Mysen
Norway

The training video is available on www.thermosaver.no and <https://thermosaver.no/produkt/instruktion-for-usage/>. If you have any questions or concerns related to this manual or Thermosaver devices, contact us at post@thermosaver.no

Vennligst referer til elektronisk RU på <https://thermosaver.no/produkt/instruktion-for-usage/>

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