	Document: PS-031 Product Data Sheet Vapour Barrier		
	Version: 1.0	Document No: PS-031	Page: 1 of 4
Prepared by: Ketil Thorsen	Approved by: Ketil Thorsen	Date of approval: 22.12.2021	




PRODUCT DATA SHEET

Gemini 300-3

Vapour barrier for Thermosaver Hypothermic Rescue Bag.

The fabric in the vapour barrier is polypropylene with a breathable barrier (trilaminate). Welded on in a rectangular zone in the middle of the Gemini 300 is a moisture-absorbing medical sheet that can absorb about 10 liters of liquid. For easy wrapping of a patient during use, the vapour barrier is marked with the numbers 1 - 2 - 3 to ensure the correct sequence when wrapping the patient. The trilaminate is suitable for lifting the patient onto a hospital bed. When lifting, it is recommended to roll up the vapour barrier at the edges for a good grip. The vapour barrier is for single use only and should be destroyed according to medical guidelines after use. The vapour barriers are individually vacuum-packed and sold in packages of three.


	Document: PS-031 Product Data Sheet Vapour Barrier		
	Version: 1.0	Document No: PS-031	Page: 2 of 4
Prepared by: Ketil Thorsen	Approved by: Ketil Thorsen		Date of approval: 22.12.2021



PRODUCT DATA SHEET


Jona 300-3

Vapour barrier for Thermosaver Hypothermic Rescue Bag.

	Document: PS-031 Product Data Sheet Vapour Barrier		
	Version: 1.0	Document No: PS-031	Page: 3 of 4
Prepared by: Ketil Thorsen	Approved by: Ketil Thorsen		Date of approval: 22.12.2021

The vapor barrier is a custom cut ASAP Leana B moisture-absorbent sheet with an absorbency of 10 l per square meter. The vapour barrier absorbs over the entire area of the sheet. The vapour barrier is marked with the numbers 1 - 2 - 3 to ensure the correct sequence when wrapping the patient. The vapour barrier is also suitable for moving the patient from the rescue bag to a hospital bed. When lifting, it is recommended to roll up the vapour barrier at the edges for a good grip. The vapour barrier is for single use only and must be destroyed according to medical guidelines. Jona 300 is Nordic Ecolabelled. The vapour barriers are individually vacuum-packed and sold in packages of three.


	Gemini 300-3	Jona 300-3
Compressed L/W/H cm	32/32/5,5	32/32/8
In Use L/W/H cm	275/192/0,2	275/192/0,2
Weight g (g per unit)	1300 (433)	1500 (500)
Colour	Yellow/blue and white absorbtion layer	Light blue
Fabric	Non-woven polypropylene and 70 % non-woven viscose 30 % polyester	70% Non-woven viscose 30 % polyester
Absorbtion capacity	ca 10 l	ca 30 l

	Document: PS-031 Product Data Sheet Vapour Barrier		
	Version: 1.0	Document No: PS-031	Page: 4 of 4
Prepared by: Ketil Thorsen	Approved by: Ketil Thorsen		Date of approval: 22.12.2021

Revision History

Revision number	Date	Reason for change	Description of change
1	See published date above	n/a	Initial release

Approval

Approval	Date	Signature
Name: Ketil Thorsen CEO Name: Hiwa Hosseinzadeh OM	22-DEC-2021	KT:  HH: 